









URINE CONTAINER GUIDE

	<p>Urinalysis (Clean Catch, Cath)</p>	<ul style="list-style-type: none"> • Urinalysis - • Urinalysis – Macroscopic • Urinalysis – Microscopic <p>Any Urinalysis testing, with or without reflex.</p>
	<p>Urine Culture (Clean Catch, Cath)</p>	<ul style="list-style-type: none"> • Urine Culture - Minimum of 3 mL <p>Note: Tube should always accompany a yellow tube when a Urinalysis is ordered, regardless of reflex criteria.</p>
	<p>Urine Chemistry (Clear tube)</p>	<ul style="list-style-type: none"> • Microalbumin/Creatinine/Protein • Sodium, potassium, chloride • Osmolality • Rapid Urine Drug Screen • Urine hCG • Urine Cotinine
	<p>CTNG/Trichomonas >14 years old (Dirty, voided)</p>	<ul style="list-style-type: none"> • Chlamydia and Gonorrhea • Trichomonas Vaginalis <p>For patients <14 years of age, Aptima urine container MUST be used.</p>
	<p>Urine Collection Cup</p>	<ul style="list-style-type: none"> • Controlled Substance Monitoring- 30 ml minimum • Short volume urines - Minimum of 3 mL unless peds, neonate, nephrostomy, kidney, or surgical collection <p>Urine cup lids should be parafilmmed by clinic, prior to transport.</p>
	<p>24-hour Urine Jug</p>	<ul style="list-style-type: none"> • 24-hour urine testing <p>Stability and additive requirements MUST be reviewed across all orders prior to giving to patient for collection. These can be found in the St. Luke's Lab Test Catalog.</p>

This list is not all inclusive, but to help ensure specimen stability and integrity, and instrument readiness.

To request tubes and supplies please fax a *Supply Request* form to MedLabs at 319-861-6880.

For further specimen requirements, please visit the St Lukes test catalog at <https://stlukescedarrapids.testcatalog.org> or contact MedLabs at 319-368-5650 or St Luke's lab 319-369-7311